Stap 8

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CHECKLIST CHECKLISTE CHECKLIST

CHECKLIST - ENGLISH

- Step A** Make sure to activate your test online at www.homediq.com/activate. If your test is not activated, the laboratory cannot process the test.
- Step B** Be sure to follow the step-by-step instructions in this manual and to fill all tubes.
- Step C** Place the sealed collection tubes in the supplied plastic pouch and close the bag using the plastic adhesive edge. Place the pouch in the test box and seal it using the white adhesive strip on the inside of the lid. Return the test by post to the address stated on the box.
- Step D** Once you have collected the test material, you should immediately mail the test kit to the return address on the test box. *If the temperature outside is under 3°C or above 25°C, please take the package to a post point rather than dropping it in a mailbox.*

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NEXT STEPS NÄCHSTE SCHRITTE VOLGENDE STAPPEN

ENGLISH

Step 1
Once the lab has received your test sample, they will process it.

Step 2
Your test results will be reviewed by our medical team.

Step 3
We will contact you immediately once your test results are ready.

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Scan the QR code to view our instructional video.

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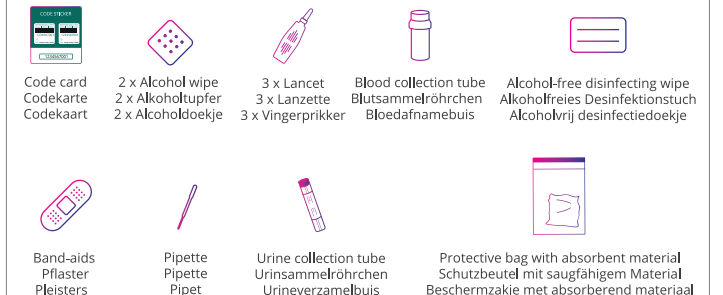
QUESTIONS? FRAGEN? VRAGEN?

homediq.com/faq
homediq.com/de/faq
homediq.com/nl/faq support@homediq.com

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(EN)

PREPERATION

Step 1  In the test box you will find a green code card. Write your date of birth on the barcode stickers and place them vertically on the collection tubes.

Step 2  **IMPORTANT:** Go to www.homediq.com/activate and activate your test with the 12-digit ID number printed on the green code card and the barcode stickers you have just placed on the collection tubes. If you do not do this, the lab will not be able to process your test sample.

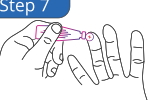
TAKE YOUR BLOOD TEST

Step 3  **IMPORTANT: A.** Make sure to drink adequate water 30 minutes before taking the blood sample. **B.** Wash your hands with warm water for 30 seconds and dry them well. **C.** Next, hold your hands against a cup of warm water for 2 minutes to stimulate blood circulation. **D.** Finally, vigorously swing your arms at your sides 8-10 times.

Step 4  Carefully unscrew the cap of the collection tube.

Step 5  Twist the cap of the lancet to remove it. The lancet is now ready to use. Each lancet can only be used once.

Step 6  Select the site of the finger prick. The best location for blood collection is both the middle and ring fingers, on the fingertip and slightly to the side. Use the included alcohol wipes to properly disinfect the finger. Let the finger dry thoroughly. For other blood collection tips, please visit our website: www.homediq.com/en/blood-collection-tips.

Step 7  Place the lancet against the disinfected finger. Press the lancet firmly until you hear a click. If you have trouble with this, ask someone else to help you. Wipe off the first drop of blood that appears with the included alcohol-free disinfecting wipe.

Step 8  Stand up and keep your hand below chest height. Gently massage the finger from the palm towards the fingertip. Drip the blood droplets into the tube or scoop them up with the edge of the collection tube. If not enough blood can be collected, we recommend that you clean the finger with the included alcohol-free disinfecting wipe.

IMPORTANT: If possible, fill the tube to the top line (600 µL) to maximize the chance of success. If that doesn't work, be sure to collect at least 500 µL, as this is the minimum amount the lab needs to analyze the sample.

Step 9  Put the cap back on the blood collection tube and press it until you hear a distinct click. **IMPORTANT:** Gently tilt the sealed tube 8 to 10 times.

TAKE YOUR URINE TEST

Step 10  **A.** Do not urinate for at least 1 hour before collecting your sample. For best results, sample the first urine of the day. **B.** Do not take this test if you are currently taking antibiotics or have taken antibiotics in the past 7 days.

Step 11  Prepare the pipette and urine collection tube.

Step 12  Use a dry, clean cup to collect the first part of the first urine of the day.

Step 13  Collect the first part of the (morning) urine in the cup until the cup is half full (a minimum of 20-25ml). You can discard the rest of the urine into the toilet.

Step 14  Remove the cap from the urine collection tube and make sure that the liquid inside the tube does not spill. Use the supplied pipette to deposit urine from the cup into the urine collection tube. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine collection tube label.

Step 15  Double-check that the urine level is neatly positioned between the fill lines. Close the tube tightly with the cap provided.

FINISH

Step 16  Place the sealed collection tubes in the supplied protective bag and close it using the plastic adhesive edge.

Step 17  Next, place the protective bag containing the tubes in the test box and seal it using the white adhesive strip on the inside of the lid.

Step 18  Return the test by post to the address stated on the box. The test sample fits in the regular letterbox. If you decide to drop off the package at a post office, we request that you do NOT add track & trace to the shipment, this will delay the shipping process. The costs of return shipping have been prepaid. *Your test sample will only be processed if it has been activated via our website.*

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