CHECKLIST - ENGLISH

- Step A** Make sure to activate your test online at www.homediq.com/activate. If your test is not activated, the laboratory cannot process the test.
- Step B** Be sure to follow the step-by-step instructions in this manual and to fill all tubes.
- Step C** Place the sealed collection tubes in the supplied plastic pouch and close the bag using the plastic adhesive edge. Place the pouch in the test box and seal it using the white adhesive strip on the inside of the lid. Return the test by post to the address stated on the box.
- Step D** Once you have collected the test material, you should immediately mail the test kit to the return address on the test box. *If the temperature outside is under 3°C or above 25°C, please take the package to a post point rather than dropping it in a mailbox.*

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ENGLISH

Step 1
Once the lab has received your test sample, they will process it.

Step 2
Your test results will be reviewed by our medical team.

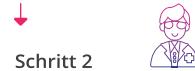
Step 3
We will contact you immediately once your test results are ready.

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QUESTIONS? FRAGEN? VRAGEN?

homediq.com/faq support@homediq.com
homediq.com/de/faq
homediq.com/nl/faq

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INSTRUCTIONS - ENGLISH

(EN)

Step 1  In the test box you will find a green code card. Write your date of birth on the barcode stickers and place them vertically on the collection tubes.

Step 2  **IMPORTANT:** Go to www.homediq.com/activate and activate your test with the 12-digit ID number printed on the green code card and the barcode stickers you have just placed on the collection tubes. If you do not do this, the lab will not be able to process your test sample.

Step 3  **IMPORTANT: A.** Make sure to drink adequate water 30 minutes before taking the blood sample. **B.** Wash your hands with warm water for 30 seconds and dry them well. **C.** Next, hold your hands against a cup of warm water for 2 minutes to stimulate blood circulation. **D.** Finally, vigorously swing your arms at your sides 8-10 times.

Step 4  Unscrew the caps of the 2 collection tubes. Then, twist the cap of the lancet to remove it. The lancet is now ready to use. Each lancet can only be used once.

Step 5  Select the site of the finger prick. The best location for blood collection is both the middle and ring fingers, on the fingertip and slightly to the side. Use the included alcohol wipes to properly disinfect the finger. Let the finger dry thoroughly. For other blood collection tips, please visit our website: www.homediq.com/en/blood-collection-tips.

Step 6  Place the lancet against the disinfected finger. Press the lancet firmly until you hear a click. If you have trouble with this, ask someone else to help you. Wipe the first drop of blood that appears with the included alcohol-free disinfecting wipe.

Step 7  Start by completely filling the tube with the **yellow cap**. When this is filled, you can fill the tube with the **purple cap**. Stand up and keep your hand below chest height. Gently massage the finger from the palm towards the fingertip. Drip the blood droplets into the tube or scoop them up with the edge of the collection tube. If not enough blood can be collected, repeat steps 6-8 with another finger using the additional lancets.
IMPORTANT: Fill both tubes to the top line (600 µL for yellow and 500 µL for purple) as this is the minimum the lab needs to analyze the sample.

Step 8  Place the caps back on the blood collection tubes and press them until you hear a distinct click. Make sure to put the yellow cap on the yellow tube and the purple cap on the purple tube. **IMPORTANT:** Gently tilt the sealed tubes 8 to 10 times.

Step 9  Place the sealed collection tubes in the supplied protective bag and close it using the plastic adhesive edge.

Step 10  Next, place the protective bag containing the tubes in the test box and seal it using the white adhesive strip on the inside of the lid.

Step 11  Return the test by post to the address stated on the box. The test sample fits in the regular letterbox. If you decide to drop off the package at a post office, we request that you do NOT add track & trace to the shipment, this will delay the shipping process. The costs of return shipping have been prepaid. *Your test sample will only be processed if it has been activated via our website.*

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