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## CHECKLIST CHECKLISTE CHECKLIST

### CHECKLIST - ENGLISH

- Step A**  Make sure to activate your test online at [www.homediq.com/activate](http://www.homediq.com/activate). If your test is not activated, the laboratory cannot process the test.
- Step B**  Be sure to follow the step-by-step instructions in this manual and to provide all requested samples.
- Step C**  Place the sample materials in the supplied plastic pouch and close the bag using the plastic adhesive edge. Place the pouch in the test box and seal it using the white adhesive strip on the inside of the lid. Return the test by post to the address stated on the box.
- Step D**  Once you have collected the test material, you should immediately mail the test kit to the return address on the test box. *If the temperature outside is under 3°C or above 25°C, please take the package to a post point rather than dropping it in a mailbox.*

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## QUESTIONS? FRAGEN? VRAGEN?

 [homediq.com/faq](http://homediq.com/faq) [support@homediq.com](mailto:support@homediq.com)  
[homediq.com/de/faq](http://homediq.com/de/faq)  
[homediq.com/nl/faq](http://homediq.com/nl/faq)

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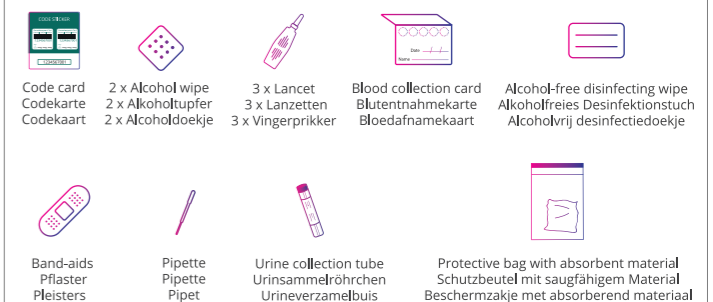
**PREPARATION**

**Step 1**  In the test box you will find a green code card. Write your date of birth on the barcode stickers and place them on the collection tube and DBS card.

**Step 2**  **IMPORTANT:** Go to [www.homediq.com/activate](http://www.homediq.com/activate) and activate your test with the 12-digit ID number printed on the green code card and the barcode stickers you have just placed on the collection tube and DBS card. If you do not do this, the lab will not be able to process your test sample.

**TAKE YOUR BLOOD TEST**

**Step 3**  **IMPORTANT: A.** Make sure to drink adequate water 30 minutes before taking the blood sample. **B.** Wash your hands with warm water for 30 seconds and dry them well. **C.** Next, hold your hands against a cup of warm water for 2 minutes to stimulate blood circulation. **D.** Finally, vigorously swing your arms at your sides 8-10 times.

**Step 4**  Open the blood collection card so that the 5 circles are visible. Open the alcohol-free disinfecting wipe and band-aids. Then, twist the cap of the lancet to remove it. The lancet is now ready to use. Each lancet can be used only once.

**Step 5**  Select the site of the finger prick. The best location for blood collection is both the middle and ring fingers, on the fingertip and slightly to the side. Use the included alcohol wipes to properly disinfect the finger. Let the finger dry thoroughly. For other blood collection tips, please visit our website: [www.homediq.com/en/blood-collection-tips](http://www.homediq.com/en/blood-collection-tips).

**Step 6**  Place the lancet against the disinfected finger. Press the lancet firmly until you hear a click. If you have trouble with this, ask someone else to help you. Wipe the first drop of blood that appears with the included alcohol-free disinfecting wipe.

**Step 7**  Stand up and keep your hand below chest height. Gently massage the finger from the palm towards the puncture site so that another large blood droplet forms. Hold the finger over the blood collection card and allow the blood droplet to fall onto one of the marked circles without touching the card. Repeat this procedure for the remaining 4 circles. **IMPORTANT:** If the blood droplets do not fall from your finger independently, carefully bring the card into contact with the blood droplet without touching the card with your finger. The 5 circles should be filled as completely as possible within the markings (at least two-thirds full). If necessary, use the second lancet to repeat steps 5-7 with another finger.

**Step 8**  Place the open blood collection card in a dry place protected from sunlight and leave it to dry for 3 to 4 hours. **IMPORTANT:** Do not place the card on a radiator or other heat source.

**TAKE YOUR URINE TEST**

**Step 9**  **A.** Do not urinate for at least 1 hour before collecting your sample. For best results, sample the first urine of the day. **B.** Do not take this test if you are currently taking antibiotics or have taken antibiotics in the past 7 days.

**Step 10**  Prepare the pipette and urine collection tube.

**Step 11**  Use a dry, clean cup to collect the first part of the first urine of the day.

**Step 12**  Collect the first part of the (morning) urine in the cup until the cup is half full (a minimum of 20-25ml). You can discard the rest of the urine into the toilet.

**Step 13**  Remove the cap from the urine collection tube and make sure that the liquid inside the tube does not spill. Use the supplied pipette to deposit urine from the cup into the urine collection tube. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine collection tube label.

**Step 14**  Double-check that the urine level is neatly positioned between the fill lines. Close the tube tightly with the cap provided.

**FINISH**

**Step 15**  Place the sample materials in the supplied protective bag and close it using the plastic adhesive edge.

**Step 16**  Next, place the protective bag containing the samples in the test box and seal it using the white adhesive strip on the inside of the lid.

**Step 17**  Return the test by post to the address stated on the box. The test sample fits in the regular letterbox. If you decide to drop off the package at a post office, we request that you **DO NOT ADD TRACK & TRACE** to the shipment, this will delay the shipping process. The costs of return shipping have been prepaid. *Your test sample will only be processed if it has been activated via our website.*

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