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CHECKLIST CHECKLISTE CHECKLIST

CHECKLIST - ENGLISH

- Step A** Make sure to activate your test online at www.homediq.com/activate. If your test is not activated, the laboratory cannot process the test.
- Step B** Be sure to follow the step-by-step instructions in this manual and to provide all requested samples.
- Step C** Place the sample materials in the supplied plastic pouch and close the bag using the plastic adhesive edge. Place the pouch in the test box and seal it using the white adhesive strip on the inside of the lid. Return the test by post to the address stated on the box.
- Step D** Once you have collected the test material, you should immediately mail the test kit to the return address on the test box. *If the temperature outside is under 3°C or above 25°C, please take the package to a post point rather than dropping it in a mailbox.*

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NEXT STEPS NÄCHSTE SCHRITTE VOLGENDE STAPPEN

ENGLISH

Step 1  Once the lab has received your test sample, they will process it.

Step 2  Your test results will be reviewed by our medical team.

Step 3  We will contact you immediately once your test results are ready.

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Scan the QR code to view our instructional video.

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QUESTIONS? FRAGEN? VRAGEN?

homediq.com/faq
homediq.com/de/faq
homediq.com/nl/faq

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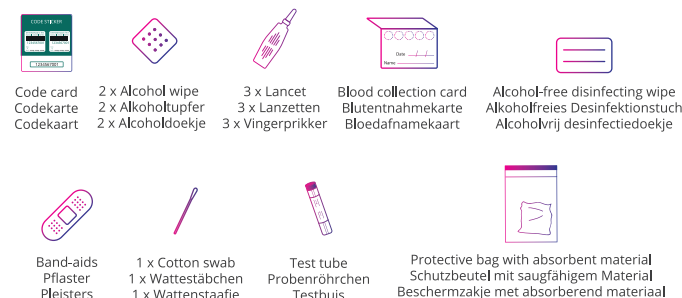
PREPARATION

Step 1 In the test box you will find a green code card. Write your date of birth on the barcode stickers and place them on the collection tube and DBS card.

Step 2 **IMPORTANT:** Go to www.homediq.com/activate and activate your test with the 12-digit ID number printed on the green code card and the barcode stickers you have just placed on the collection tube and DBS card. If you do not do this, the lab will not be able to process your test sample.

TAKE YOUR BLOOD TEST

Step 3 **IMPORTANT: A.** Make sure to drink adequate water 30 minutes before taking the blood sample. **B.** Wash your hands with warm water for 30 seconds and dry them well. **C.** Next, hold your hands against a cup of warm water for 2 minutes to stimulate blood circulation. **D.** Finally, vigorously swing your arms at your sides 8-10 times.

Step 4 Open the blood collection card so that the 5 circles are visible. Open the alcohol-free disinfecting wipe and band-aids. Then, twist the cap of the lancet to remove it. The lancet is now ready to use. Each lancet can be used only once.

Step 5 Select the site of the finger prick. The best location for blood collection is both the middle and ring fingers, on the fingertip and slightly to the side. Use the included alcohol wipes to properly disinfect the finger. Let the finger dry thoroughly. For other blood collection tips, please visit our website: www.homediq.com/en/blood-collection-tips.

Step 6 Place the lancet against the disinfected finger. Press the lancet firmly until you hear a click. If you have trouble with this, ask someone else to help you. Wipe the first drop of blood that appears with the included alcohol-free disinfecting wipe.

Step 7 Stand up and keep your hand below chest height. Gently massage the finger from the palm towards the puncture site so that another large blood droplet forms. Hold the finger over the blood collection card and allow the blood droplet to fall onto one of the marked circles without touching the card. Repeat this procedure for the remaining 4 circles. **IMPORTANT:** If the blood droplets do not fall from your finger independently, carefully bring the card into contact with the blood droplet without touching the card with your finger. The 5 circles should be filled as completely as possible within the markings (at least two-thirds full). If necessary, use the second lancet to repeat steps 5-7 with another finger.

Step 8 Place the open blood collection card in a dry place protected from sunlight and leave it to dry for 3 to 4 hours. **IMPORTANT:** Do not place the card on a radiator or other heat source.

OPTION A: VAGINAL SWAB

Step A Wash your hands with warm water for 30 seconds.

Step B Remove the collection swab from the package and ensure you do not touch the cotton bud end. Hold the swab with your thumb and forefinger.

Step C Sit or stand and insert the swab about 4 to 5 cm into your vagina (similar to how you would insert a tampon).

Step D Gently rotate the swab for 25 to 30 seconds, then remove the swab from your vagina without touching your skin.

Step E Uncap the sample collection tube and keep it upright. Do not pour out the clear liquid. Place the swab into the tube.

Step F Carefully break the stick of the swab along the indicated breakline. Then secure the cap on the test tube with the cotton swab inside it.

OPTION B: ANAL SWAB

Step A Remove the collection swab from the package and ensure you do not touch the cotton bud end. Hold the swab with your thumb and forefinger.

Step B Squat or lift one leg onto a raised surface. Insert the swab 3-4 cm into your anus.

Step C Gently rotate the swab for 10 seconds.

Step D Uncap the sample collection tube and keep it upright. Do not pour out the clear liquid. Place the swab into the tube.

Step E Align the score line with the top edge of the collection tube and carefully break the shaft of the swab. Place the cap back on the test tube and tighten.

OPTION C: THROAT SWAB

Step A Wash your hands with warm water for 30 seconds.

Step B Remove the collection swab from the package and ensure you do not touch the cotton bud end. Hold the swab with your thumb and forefinger.

Step C Stand in front of the mirror and open your mouth wide. Touch the end of the swab to the following areas of the throat: tonsils, posterior wall, and uvula. Repeat 5-6 times.

Step D Uncap the sample collection tube and keep it upright. Do not pour out the clear liquid. Place the swab into the tube.

Step E Align the score line with the top edge of the collection tube and carefully break the shaft of the swab. Place the cap back on the test tube and tighten.

SHIPPING PROCESS

Step 9 Place the sample materials in the supplied protective bag and close it using the plastic adhesive edge.

Step 10 Next, place the protective bag containing the samples in the test box and seal it using the white adhesive strip on the inside of the lid.

Step 11 Return the test by post to the address stated on the box. The test sample fits in the regular letterbox. If you decide to drop off the package at a post office, we request that you **DO NOT ADD TRACK & TRACE** to the shipment, this will delay the shipping process. The costs of return shipping have been prepaid. *Your test sample will only be processed if it has been activated via our website.*

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