

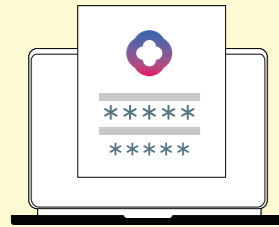
WIE SIE IHRE STUHLPROBE SAMMELN [DE] / HOW TO COLLECT YOUR STOOL SAMPLE [EN] / HOE KUN JE JOUW SAMPLE VERZAMELEN [NL]

1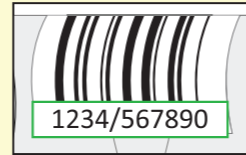

- NL BELANGRIJK:** Activeer uw testkit op www.homediq.com/activeer anders kan het laboratorium uw testmonster niet verwerken.
- EN IMPORTANT:** Activate your test kit at www.homediq.com/activate otherwise the lab will not be able to process your test sample.
- DE WICHTIG:** Aktivieren Sie Ihr Testkit unter www.homediq.com/aktivieren andernfalls kann das Labor Ihre Testprobe nicht verarbeiten.

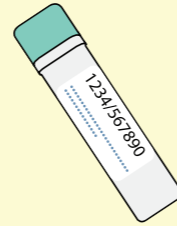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Once the laboratory has received your test sample, they will process it. [EN]

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Your test results will be reviewed by our medical team. [EN]

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We will contact you immediately once your test results are ready. [EN]

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This is your test kit for Homed-IQ's Colon Cancer Screening Test. Taking the sample for this test only takes a few minutes. This test helps to detect colon cancer early. The earlier colon cancer is found, the easier it is to treat. This manual contains the instructions needed to successfully complete your test. [EN]

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For Terms and Conditions for the NHS Blood and Transplant Convalescent Plasma clinical trial, please click here.

Terms & Conditions - Homed-IQ Europe B.V.

pending or cancelled. It is your responsibility to resolve any issue we encounter in order to place your order. Please note that it is your responsibility to ensure that you have chosen the method and that you authorize us for our third-party payment processor(s) to charge your order. The payment method for the full amount of your order.

5. THE DELIVERY PROCESS
5.1. We aim to ship the product to the delivery address provided to us within a maximum of seven working days after the order confirmation. If the delivery address changes after we send you the order confirmation, the product will be delivered to the original address.
5.2. The delivery dates consist of: (i) the delivery date of the product; (ii) date for providing the test information; (iii) the date for receiving the test results. The delivery dates are approximate Homed-IQ estimates for delivery of the product, not a guarantee of delivery. Incomplete or missing information may result in non-compliance with the planned delivery dates or for reporting to the relevant authorities.
5.3. Product-related risks pass to you once the product has been delivered to the delivery address stated in the order confirmation.
5.4. Please email within 7 days of receiving the delivered product(s) if you have received any live/damaged products or possibly missing items. You can contact info@homediq.com and include at least the following information in your email: the order confirmation number, the Homed-IQ unique activation code and the order ID of the missing or defective product(s).
5.5. We will contact you as soon as possible and send you the product if it is damaged or defective or if certain items are missing from the Homed-IQ test kit otherwise we cannot send you a replacement.

6. CANCELLATION
6.1. You may cancel your order at any time up to 14 days from the day you receive the product to cancel the contract without giving any reason. The cancellation right expires after 14 days.
6.2. Notify us within the accepted time frame of your decision to cancel by mail or email at info@homediq.com. You must then return the product to us within a maximum of 14 days of receipt of the product. We reimburse you with the same payment method you used for the first payment unless you have used a different method. The return of the product will be charged as a result of such reimbursement. As mentioned earlier, the costs of returning the product will be at your own expense.
7. OUR SERVICES
7.1. In case you receive the product by mail or purchase the product in-store from one of the pharmacies and/or retailers, you do not need to register the test. If you do not link it to your personal account, you do not need to register the test. You will not have access to the test information, the test results, the test reactive or detected test, can be free of infectious disease. Any positive, reactive, or definitive diagnosis through a confirmed physician through a confirmation of test.
7.2. Any breach of your obligations as described in the Terms and Conditions may constitute a breach of contract and you may be liable for damages and/or compensation. By using our year 1 product, you warrant and confirm that you are legally able to enter into binding contracts and that you are at least 18 years old.
7.3. All products and follow-up tests performed by a medical professional are required to form a definitive diagnosis.
7.4. The incubation period is the period from exposure to the person's body. Different viruses and infections have different incubation times and can take several months to be detected. You should be aware of this and use the test before the relevant incubation period has completely expired (the incubation period is not developed; or 3) if you may have been exposed to any other infection in the meantime.
7.5. You undertake and agree: steps in the supplied instructions for use that are included with the product; to return the sample within the timeframe set out in the product; and to provide us, when requested, with the information or proof of identity that we may require from you to correct or delete any information and to ensure that it is accurate and that you have given your consent to use the website, products and services only for the purposes for which they were intended and that is outlined within the Terms and Conditions. In addition, you further agree not to make a complaint or ask any questions regarding these Terms and Conditions, not to offend or frighten any of our employees or our employees. You also agree not to use the products and/or services for resale or in any way that violates our intellectual property rights or other applicable laws and regulations. You acknowledge and agree that when you place an order for the services through this website including purchasing one or more products through this website, you agree to be bound by the Terms and Conditions and to accept any advice from us.

8. SUBMISSION OF THE TEST SAMPLE
8.1. Please note that when submitting a sample, you must fill the test tube to the indicated level. If you do not provide enough test material, the laboratory cannot process your sample which means you will not receive a test result in the user manual. Please read this carefully in the user manual. Please read this carefully to make sure you complete the steps of the test. If you do not complete the steps, we will not be able to process the test. You then place the test sample, after collection, in a prepaid envelope and you must immediately (read: immediately after collection) return it to the laboratory. Do not use other containers or other methods of transport. Do not use any other containers or other methods of transport. Do not use any other containers or other methods of transport.
9. SETTING UP YOUR ACCOUNT
9.1. To obtain test results that we have created on an account on the website and activated the test using the unique ID, code stated on the product.
9.2. The user agrees to create only one account with us through the website and not to duplicate a user account. When you create an account, we will ask for your personal information and the correct contact information and to answer the questionnaire carefully as it will be used to support our medical team.
9.3. To create an account, go to 'login' at the top of the menu on the homepage of the website.

10. TEST ACTIVATION
10.1. All tests must be activated by the customer before use. Failure to activate one's test may result in Homed-IQ being unable to process a test result.
10.2. The customer is responsible for reading all test instructions and ensuring their test is correctly activated. Homed-IQ is not liable to reimburse customers for incorrectly activated tests.
11. TEST INFORMATION
11.1. You will receive an email notification when the test results have been made available when the account. The test results are written in plain text and you can access the test results and the outcome of the test. You may receive a notification of non-reactive, please refer to section 18 of the Terms and Conditions and the activation times.
11.2. A confirmatory test from a qualified physician is necessary if you have received a positive result. With regard to user health test information indicates a positive, reactive, detected or increased result, our services includes a written report and you will be contacted by our support team in addition to this explanation, we do advise the user to contact his or her doctor.
11.3. If you purchase products and services and receive test information within an incubation period, you will need to wait for the test results or through your doctor, if you are concerned or have any other questions, you should consult your own physician.
12. DISCLAIMERS AND EXCLUSION OF LIABILITY
12.1. We, our affiliates and their respective directors, employees and suppliers, make no warranties or representations about the content (including user content) of the Homed-IQ website, products and services, including but not limited to its accuracy, completeness or reliability. We do not warrant the accuracy, completeness or reliability of the information provided by the user.
12.2. You, the consumer, "user", have certain legal rights. The disclaimers, exclusions and limitations of liability under these Terms and Conditions apply to you. Nothing in these Terms and Conditions will attempt to exclude or limit your liability that cannot be excluded under applicable law.
12.3. The products and services are provided "as is", without any form of warranty. We hereby disclaim all warranties, including, but not limited to, implied warranties of merchantability, non-infringement of third party rights, and fitness for a particular purpose.
12.4. In addition to the above-mentioned warranties, we make no warranty, whether in a written, special or incidental damages, including damages for loss of data, loss of reputation, lost profits, lost business, lost contracts or other losses suffered by a third party, arising out of or in connection with the products and/or services. Homed-IQ Europe B.V. or our affiliates, even if we knew or should have known the possibility of such damage.
12.5. Under no circumstances will the total liability of Homed-IQ Europe B.V. in no way liable or responsible for the amount that you have paid for the products and services, whether in contract or otherwise, exceed 100% of the amount that you have paid for the products and services.
12.6. Homed-IQ Europe B.V. does not accept any liability or commitments set out in the General Terms and Conditions will never read us to be in breach of any provision of these General Terms and Conditions.
12.7. Homed-IQ Europe B.V. is not obliged to provide the services if you have not followed the instructions regarding ordering using the product. Homed-IQ Europe B.V. is not liable for the sample in accordance with these General Terms and Conditions.
12.8. The image of a product on our website may not accurately reflect the actual product. Homed-IQ Europe B.V. is not liable for any discrepancy between the image of a product on our website and the actual product.
12.9. The provisions of this clause will remain in full force and effect upon the expiration of the General Agreement.

13. GENERAL LEGAL PROVISIONS
13.1. Homed-IQ Europe B.V. is in no way liable or responsible for the amount that you have paid for the products and services, whether in contract or otherwise, exceed 100% of the amount that you have paid for the products and services.
13.2. For these General Terms and Conditions, Dutch law applies to the construction, interpretation and in case of any dispute, our website and in case of any dispute, our website and in case of any dispute, our website and in case of any dispute, our website.
13.3. Homed-IQ Europe B.V. is not obliged to provide the services if you have not followed the instructions regarding ordering using the product. Homed-IQ Europe B.V. is not liable for the sample in accordance with these General Terms and Conditions.
13.4. The image of a product on our website may not accurately reflect the actual product. Homed-IQ Europe B.V. is not liable for any discrepancy between the image of a product on our website and the actual product.
13.5. Homed-IQ Europe B.V. is not obliged to provide the services if you have not followed the instructions regarding ordering using the product. Homed-IQ Europe B.V. is not liable for the sample in accordance with these General Terms and Conditions.
13.6. If any part of these Terms and Conditions becomes illegal, invalid, unenforceable, or otherwise unworkable, the provisions of the law or regulation that provides for that part thereof will be deemed not to form part of the contract between us. The legality, validity or enforceability of the remaining provisions of the conditions will remain in full force and effect.

Last modified on August 6, 2023

ONTLASTINGSTEST [NL]
STOOL SAMPLE TEST [EN]
STUHLTEST [DE]

Lees eerst de gebruiksaanwijzing voordat u begint met de testafname [NL]
Read the instruction manual first before you start collecting your sample [EN]
Lesen Sie zuerst die Bedienungsanleitung, bevor Sie mit dem Test beginnen [DE]

Last modified on August 6, 2023