CHECKLIST - ENGLISH

- Step A** Make sure to activate your test online at www.homediq.com/activate. If your test is not activated, the laboratory cannot process the test.
- Step B** Be sure to follow the step-by-step instructions in this manual and to fill all tubes.
- Step C** Place the sealed collection tube in the supplied plastic pouch and close the bag using the plastic adhesive edge. Place the pouch in the test box and seal it using the white adhesive strip on the inside of the lid. Return the test by post to the address stated on the box.
- Step D** Once you have collected the test material, you should immediately mail the test kit to the return address on the test box. *If the temperature outside is under 3°C or above 25°C, please take the package to a post point rather than dropping it in a mailbox.*

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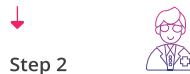
ENGLISH

Once the lab has received your test sample, they will process it.


Your test results will be reviewed by our medical team.


We will contact you immediately once your test results are ready.

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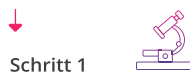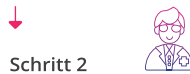QUESTIONS? **FRAGEN?** **VRAGEN?**

- homediq.com/faq
- support@homediq.com
- homediq.com/de/faq
- homediq.com/nl/faq

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
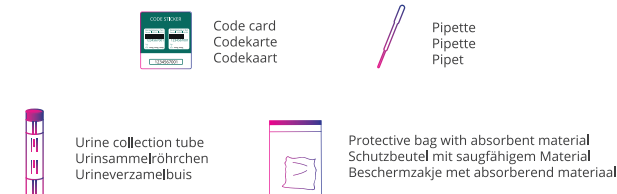
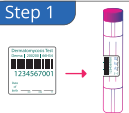
(EN)

Step 1

In the test box you will find a green code card. Write your date of birth on the barcode sticker and place it vertically on the collection tube.

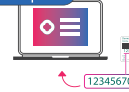Step 2

IMPORTANT: Go to www.homediq.com/activate and activate your test with the 12-digit ID number printed on the green code card and the barcode sticker you have just placed on the collection tube. If you do not do this, the lab will not be able to process your test sample.

Step 3

A. Do not urinate for at least 1 hour before collecting your sample. For best results, sample the first urine of the day. **B.** Do not take this test if you are currently taking antibiotics or have taken antibiotics in the past 7 days.

Step 4

Prepare the pipette and urine collection tube.

Step 5

Use a dry, clean cup to collect the first part of the first urine of the day.

Step 6

Collect the first part of the (morning) urine in the cup until the cup is half full (a minimum of 20-25 ml). You can discard the rest of the urine into the toilet.

Step 7

Remove the cap from the urine collection tube and make sure that the liquid inside the tube does not spill. Use the supplied pipette to deposit urine from the cup into the urine collection tube. The correct volume of urine has been added when the fluid level is between the black fill lines on the urine collection tube label.

Step 8

Double-check that the urine level is neatly positioned between the fill lines. Close the tube tightly with the cap provided.

Step 9

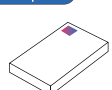
Place the sealed urine collection tube in the supplied protective pouch and close it using the plastic adhesive edge.

Step 10

Next, place the protective bag containing the tube in the test box and seal it using the white adhesive strip on the inside of the lid.

Step 11

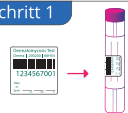
Return the test by post to the address stated on the box. The test sample fits in the regular letterbox. If you decide to drop off the package at a post office, we request that you do NOT add track & trace to the shipment, this will delay the shipping process. The costs of return shipping have been prepaid. *Your test sample will only be processed if it has been activated via our website.*

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